

Visi-Download release notes for 151019

New features

- Snipping option added via toolbar icon, including Edit/Delete and save snipping images and text via toolbar icon, and inclusion on reports
- Canned comments can be added to report comments. Create your own via Setup Wizard Canned Comments tab.
- User can enter a value in SpO2 or Pulse Rate analysis setup that causes a line to be drawn at x% SpO2 on the SpO2 plot graph (at x% bpm on the Pulse Rate plot graph) so users can visually check the curves and watch for drops or rises.
- Auto checking for updates when Visi-Download starts, so you can be sure you're always on the latest version

Improvements

- "Amplitude", "Fit to Scale" and "Auto Fit to Scale" menu options displayed if right mouse button clicked whilst in 'edit events'.
- If the measurement tool box is displayed (hold the shift key while moving the mouse), if you press a key that is associated with an event the event is added. Note - the event **must** be for the channel the measurement tool is measuring.
- Amended Black Shadow SD setup so recording duration saved for next time (previously it always defaulted to 8hrs).
- Canned comments can be used in patient details tab of Black Series, Viatom, Rad-8, TCM5 USB, TCM5 Network, Rad-7 Touch, Rad-97
- Rad-8, Rad-7 Touch, Rad-97 have more automated troubleshooting of download settings
- Signed Windows 10 device driver for Konica-Minolta, Bitmos, GreyFlash
- Viatom auto age and neck size calculated